


OASIS[®]

Bedbath Maceratable

Instructions for use


Oasis[®] Bed Bathing Washcloths are impregnated with a unique combination of skin cleansers to help remove sweat, dirt, urine and faeces from the skin. The large, soft washcloths are intended to be used individually, with one single-use cloth per anatomical region.

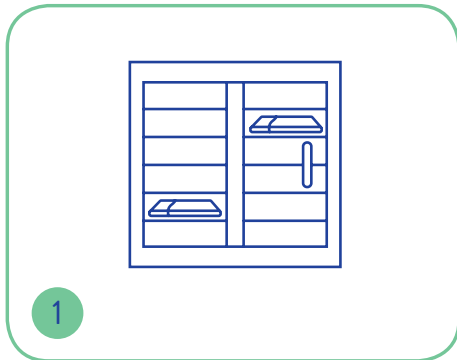


1 Remove wipes from the dedicated Oasis Warmer




2 Or use at room temperature

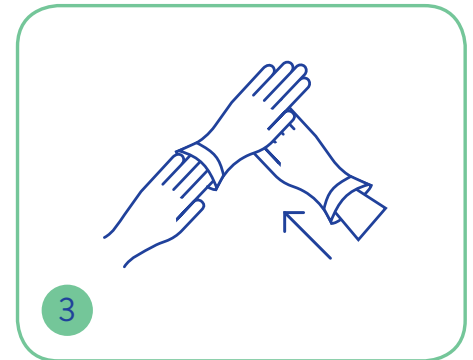


3 Apply protective gloves

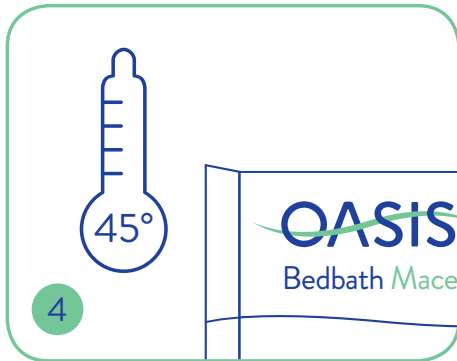


4 Note how many wipes are in each pack and check temperature

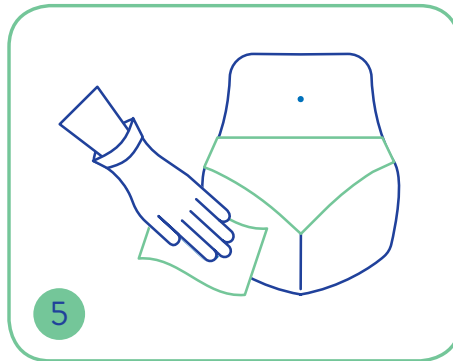

5 Use one wipe to per anatomical region

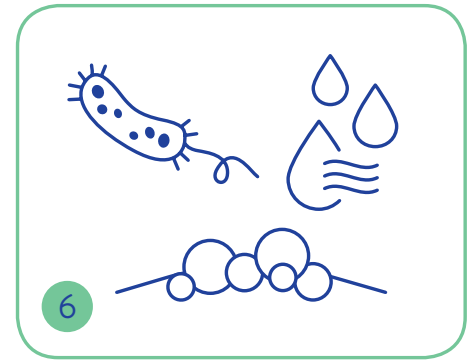

6 The wipes will cleanse and moisturise the skin

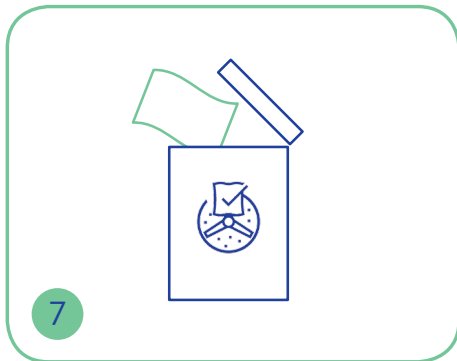

7 Dispose of used wipes in the macerator

Description	Size	Pack Qty	Carton Qty	Code
Oasis [®] Bed Bath Maceratable Scented	30 x 22cm	10	27	MBB10
Oasis [®] Bed Bath Maceratable Unscented	30 x 22cm	10	27	MBB10UP
Oasis [®] Bed Bath Maceratable Scented	30 x 22cm	8	33	MBB08
Oasis [®] Bed Bath Maceratable Unscented	30 x 22cm	8	33	MBB08UP
Oasis [®] Bed Bath Maceratable Scented	30 x 22cm	5	48	MBB05
Oasis [®] Bed Bath Maceratable Unscented	30 x 22cm	5	48	MBB05UP



Perfumed and unperfumed options available